



ComfortKone lenses provide a comfortable alignment fit to the cornea. Fitting is approached from center outward or, "from the inside-out".

Design of the posterior surface consists of a spherical curve in the central 4mm, an aspheric zone from 4mm out to 8mm and a bi-aspheric periphery.

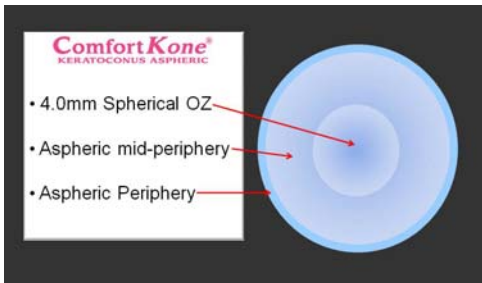

The key fitting element of the ComfortKone design is the posterior "aspheric periphery" and is represented as the "A" value. The "A" value controls the "edge lift" and provides mid-peripheral "alignment". The greater the "A" value number, the more aspheric flattening and the greater the edge lift.

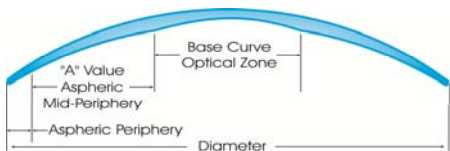

Always Ahead of the Curve

Lens Parameter Availability

- Base Curve*
4.5mm (75.00 diopters) to 8.00mm (42.00 diopters)
 - "A" Value A3 to A25 (steps of 1)
 - Diameter* 7.5mm to 9.5mm (0.1mm steps)
 - Power* Any sphere power
- *Custom Parameters available

Fitting ComfortKone

Step 1 Initial Diagnostic Lens Selection

1. Select a base curve nearest the steepest reading from keratometer readings or corneal topography.
2. Select the highest "A" value available in calculated base curve range.

Example:

52.37 @ 43 / . 57.87 @ 133 =

Steep K = 57.87

Conversion- 57.87= 5.83mm

Round to nearest .10 = 5.80mm

5.80 Base Curve/A20-
Initial Diagnostic Lens



Important Note – The initial diagnostic lens is only a starting point.

Step 2 Apply Initial Diagnostic lens and instill Fluorescein.


Note: ComfortKone diagnostic lenses contain a UV blocker. For best results, evaluate fluorescein with blue cobalt light and a **Yellow Wratten filter**.

Step 3 Evaluate Central Fluorescein Pattern

Slight Apical Clearance is preferred.

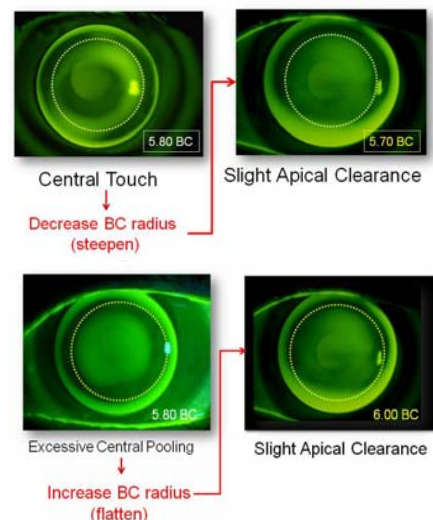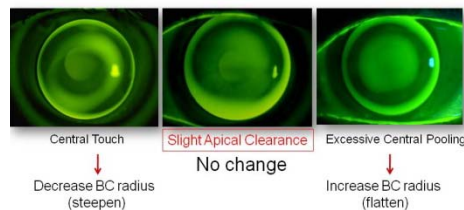

Step 4 Evaluate Edge Lift Profile Independently from Base Curve Evaluation

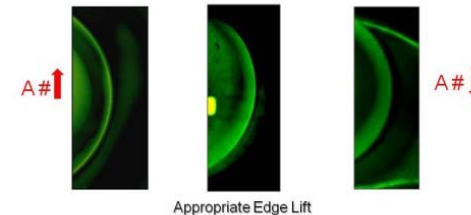

Step 5 Over-Refract

When the best fitting diagnostic lens is determined, allow the lens to settle for 3 to 5 minutes and over-refract.

Note: Vertex all Over-refraction $\pm 4.00D$

Step 6 Order Lens

Important Note – ComfortKone lenses will gravitate over the cone and do not need to position centrally to achieve excellent results. If best fitting decenters excessively, call Metro Optics consultation.

Fitting tips:

During Diagnostic Fitting:

- Make Base Curve changes in .2mm steps to evaluate effect.
- Make "A" value changes in steps of 5 "A" values to evaluate effect.